


Uterus manipulator by Cohen

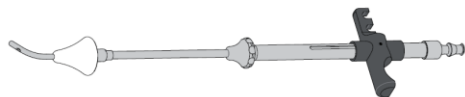

Item numbers

HY500-000		
HY500-010	HY500-020	HY500-030

Safety and responsibility

Before using the instrument

- ▶ Read and follow the Instructions for Use.
- ▶ Keep these instructions in a place where they can be easily seen for reference at a later date.

RUDOLF Medical does not assume any liability for damage as a result of improper use or incorrect preparation and care.

In such cases the warranty is rendered void.

Indication

This instrument is designed for gynecological applications and is inserted vaginally. It is used to install contrast media and to position the uterus in preparation for the subsequent intervention.

Contraindication

This instrument is not designed for use on the central nervous and circulatory system.

Risk of infection

- ▶ Prepare before use.
- ▶ Prepare the instrument before returning it to the manufacturer.

Risk of injury

- ▶ Only use original accessories
- ▶ Do not use damaged instruments and do not repair

Scope of supply


- ▶ After unpacking, check all parts for any damage caused during transit.
- ▶ In the event of complaints, contact the manufacturer.

➤ For the manufacturer's address see the last page.

i If the set is ordered, two adapters are supplied. The adapters are two different sizes.

Product description

The uterus can be positioned in preparation for the intervention using the adapter. In addition, the perforation on the adapter cone tube facilitates the instillation of contrast media for diagnostic purposes. The cone seals the cervix.

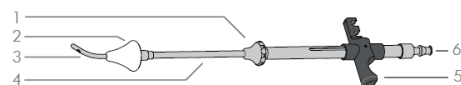

- 1 Knurled nut
- 2 Adapter
- 3 Adapter tube with perforation
- 4 Inner sheath
- 5 Handle
- 6 LL connection

Technical data

Storage and transport conditions

- ▶ Only transport and store the instrument with appropriate protection

Product lifetime


≤1000 cycles and ≤2 years

Assembly and disassembly of the instrument


Assembly of the instrument

i The size of adapter is chosen immediately prior to the surgery, according to the size of the cervix.

- ▶ Fit spring to inner sheath.

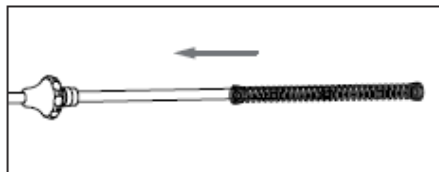


- ▶ Fit outer sheath and screw it to knurled nut.

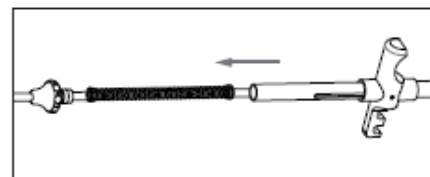


Disassembly of the instrument

- ▶ Turn adapter slightly and remove it.

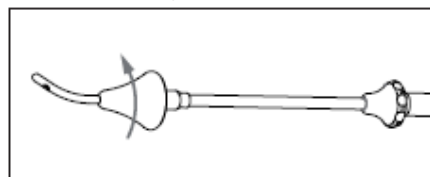

- ▶ Unscrew knurled nut and take out outer shaft with spring.

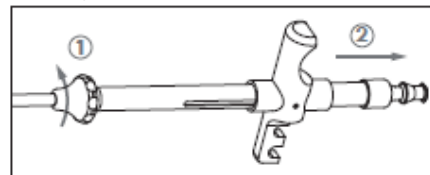


Application

WARNING

Risk of infection from non sterile instruments.


- ▶ Prepare the instrument before use
- ▶ Choose adapter according to the size of the cervix.
- ▶ Fit adapter and tighten it.

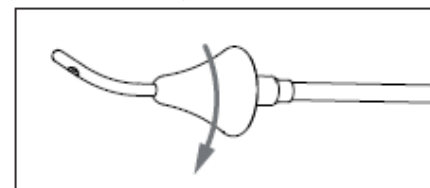


- ▶ Unscrew knurled nut slightly and turn the inner sheath into the correct position so that it is aligned with the tip of the adapter.

INSTRUCTIONS FOR USE (EN)

Uterus manipulator by Cohen

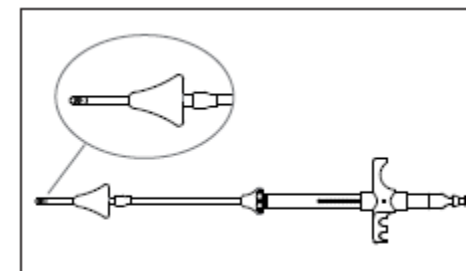

- ▶ Connect syringe containing contrast media to the LL connection.
- ▶ Insert instrument into the cervical os.
- ▶ Position the adapter as required and position the uterus
- ▶ Insert tenaculum and carefully grasp the uterus.

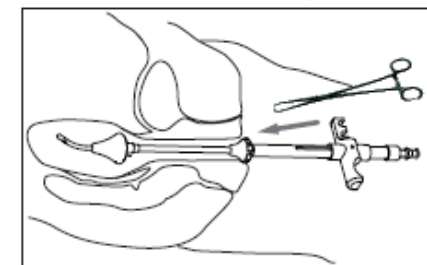

- ▶ Set the finger hole of the tenaculum into the slider.
- ↳ The instrument is securely positioned.
- ▶ Instil contrast media via the syringe.
- ▶ Prepare the instrument.

Preparation

i

The instrument must be dismantled prior to preparation.

➤ See section „Disassembly of the instrument“.

Storage

➤ Detailed information on storage and care can be found in the separate Guidelines for Care, Maintenance and Cleaning DOK#301





Disposal

Environmentally sound disposal enables valuable raw materials to be recycled.



Dispose of the device in an environmentally friendly manner in accordance with the valid hospital guidelines.

Safety information and explanation of symbols

	Follow the Instructions for use
REF	Order number
	Non sterile
	Manufacturer
	CE Mark



RUDOLF Medical GmbH + Co. KG
Zollerstr. 1, 78567 Fridingen, Germany
Tel. +49 7463 9956-0
Fax +49 7463 9956-56
sales@rudolf-med.com
www.rudolf-med.com