

**INSTRUCTIONS FOR USE (EN)  
BIOPSY, FOREIGN BODY, AND GRASPING FORCEPS AND SCISSORS  
FOR THE HYSTEROSCOPY**



RUDOLF Medical GmbH + Co. KG  
Zollerstrasse 1, 78567 Fridingen an der Donau,  
Germany  
Tel. +49 7463 9956-0  
Fax +49 7463 9956-56  
[sales@RUDOLF-med.com](mailto:sales@RUDOLF-med.com)  
[www.RUDOLF-med.com](http://www.RUDOLF-med.com)



D0819 / Rev C / ACR00413 / 2023-01-10

**PLEASE READ BEFORE REPROCESSING AND KEEP IT IN A SAFE PLACE**

**PRODUCT**

These instructions for use refer to the assembly and disassembly of the RUDOLF Medical biopsy, foreign body, and grasping forceps and the scissors for the hysteroscopy that are intended for the removal of sample tissue during hysteroscopy.

You are receiving a high-quality product whose proper handling and use are described below.



The following instructions refer to the disassembly of the instrument after surgery and assembly after the cleaning and disinfection process.

Detailed instructions for reprocessing can be found in the IFU D0462:




RUDOLF Medical instruments are delivered non-sterile and must be cleaned, disinfected, and sterilized before first use and immediately after each use. Protective caps and transport packaging must be removed beforehand.

We recommend treating the mechanical parts of the instrument with a lubricant before sterilization.


A thorough functional and visual check should be carried out before each use.

**DISASSEMBLY**

1. Loosen the screw (1) counterclockwise and remove it (2).
2. Loosen the knurled nut from the handle (1) counterclockwise and remove it.

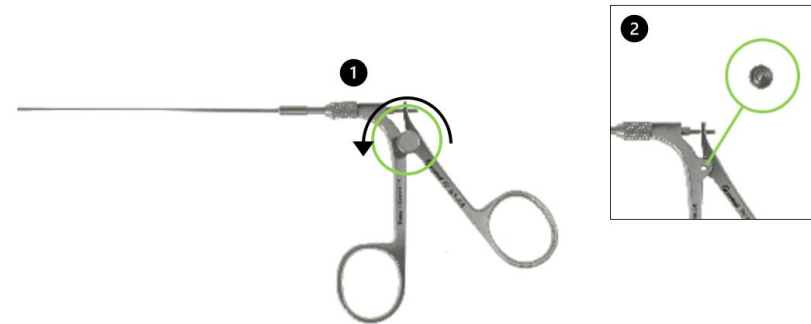


3. Remove the thumb-movable handle (2) and pull the instrument insert out of the handle (3).

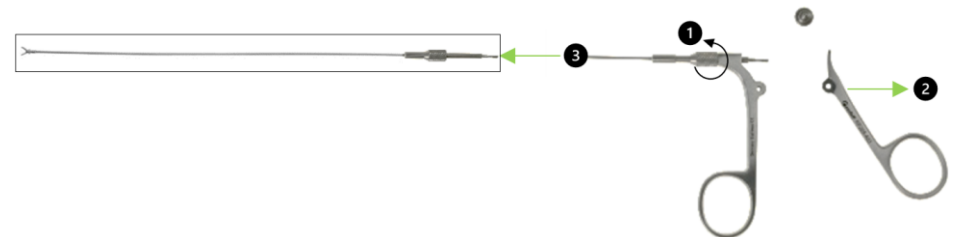

## ASSEMBLY



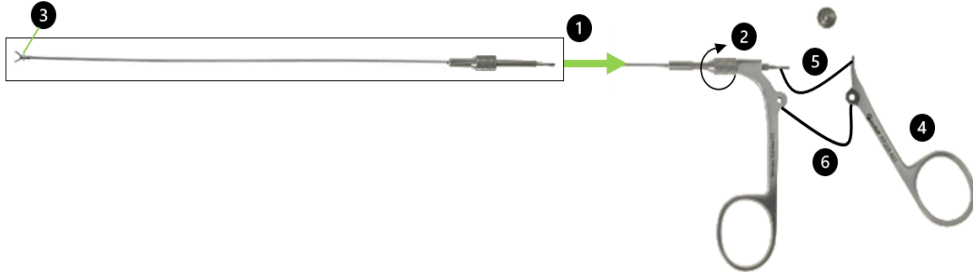








1. Insert the instrument insert (1) into the opening of the front handle, and screw on the knurled nut clockwise (2).
2. Position the working end (3) before fastening the screw.
3. Insert the thumb-movable handle (4) with the hook into the groove (5) of the instrument insert and into the screw lock (6).



2. Fasten the screw on the handle clockwise.



## SYMBOLS

	Consult instructions for use.
	Batch code
	Article no.
	No. per package
	Non-sterile
	Caution
	Manufacturer
	Date of manufacture
	CE marking according to EG directive 93/42/EWG
	Lubricate with silicon-free, biocompatible white medical oil approved for steam sterilization
	Medical Device